





PPE Guidelines for Reusing Eye Protection: Safety Glasses, Eye Shields, Goggles, and Face Shields

Here are updated recommendations for the storage and reuse of safety glasses, eye shields, goggles, and face shields. Please review the following guidelines and share with your teams:

Caregivers who use PPE need to disinfect and reuse all eye protection. This includes all types of safety glasses, eye shields, goggles, and face shields. Disinfect as outlined in the [Contact / Droplet + Eye Protection tip sheet](#), and store cleaned eye protection in a plastic bag for reuse.

Eye Protection	Reuse and Storage	Use for	Product Information	Storage Bag Information
<p>Safety Glasses / Eye Shield</p> <p>Safety Glasses</p>  <p>Eye Shield</p> 	<ul style="list-style-type: none"> • A reusable resource used by the individual caregiver. • Disinfect and store in a bag with your name written on it as supply permits. 	<p>Contact/Droplet</p>	<p>Eyeshield Assembled Tower Grab-N-Go</p> <ul style="list-style-type: none"> • Item number: 32006055 • Manufacturer: TIDI Products • MFR number: 9210A-100 <p>Eyeshield Preassemble Flip-N-Go 9300FE-50</p> <ul style="list-style-type: none"> • Item number: 32039765 • Manufacturer: TIDI Products • MFR number: LPC9300FE50 <p>Frame Safety Glasses SV100F</p> <ul style="list-style-type: none"> • Item number: 58030024 • Manufacturer: Halyard Health Inc. • MFR number: SV100F <p>Lens Replacement Safeview NS</p> <ul style="list-style-type: none"> • Item number: 58030016 • Manufacturer: Halyard Health Inc. • MFR number: SV250L 	<p>Bag Ziplock 12x15</p> <ul style="list-style-type: none"> • Item number: 97009825 • Manufacturer: Donovan Industries • MFR number: 12154M <p>Bag Ziplock 13x18</p> <ul style="list-style-type: none"> • Item number: 32052828 • Manufacturer: Medegen Medical • MFR number: MGZ2P1318

<p>Goggles</p> 	<ul style="list-style-type: none"> • A reusable resource that must be shared by all caregivers. • Disinfect and store for reuse. 	<p>Contact/Droplet Contact/Airborne</p>		<p>Bag Ziplock 12x15</p> <ul style="list-style-type: none"> • Item number: 97009825 • Manufacturer: Donovan Industries • MFR number: 12154M <p>Bag Ziplock 13x18</p> <ul style="list-style-type: none"> • Item number: 32052828 • Manufacturer: Medegen Medical • MFR number: MGZ2P1318
<p>Face Shields</p> 	<ul style="list-style-type: none"> • A reusable resource used by the individual caregiver. • Disinfect and store in a bag with your name written on it as supply permits. 	<p>Contact/Droplet Contact/Airborne</p>	<p>Shield Splash Mask with Velcro Strap</p> <ul style="list-style-type: none"> • Item number: 02000339 • Manufacturer: Encompass Techstyles • MFR number: 11000-000 <p>Shield Splash Pro Med 4505</p> <ul style="list-style-type: none"> • Item number: 75224659 • Manufacturer: Splash Shield • MFR number: 4505 <p>Shield Guardall Full Length Visor 41204</p> <ul style="list-style-type: none"> • Item number: 58026220 • Manufacturer: Halyard Health Inc. • MFR number: 41204 	<p>Bag Ziplock 12x15</p> <ul style="list-style-type: none"> • Item number: 97009825 • Manufacturer: Donovan Industries • MFR number: 12154M <p>Bag Ziplock 13x18</p> <ul style="list-style-type: none"> • Item number: 32052828 • Manufacturer: Medegen Medical • MFR number: MGZ2P1318

For other questions related to this updated PPE requirement, please speak with your manager. Please continue to work through your Supply Chain teams to get the PPE and storage bags you need, using the product information provided here.