




CONSERVE N95 RESPIRATORS, EYEWEAR, AND FACE SHIELDS


updated 04/24/20

Safety notes: When wearing an N95 respirator, do not wear face makeup of any kind as it interferes with the cleaning and disinfecting process. Facial hair around the face and chin may interfere with the seal. Remove or trim facial hair in these areas to ensure a proper seal.

Re-use eyewear and N95 respirators that are not visibly soiled or damaged

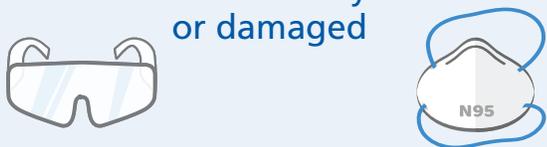

Discard eyewear and N95 respirators that are soiled, wet, damaged, and hard to see or breathe through

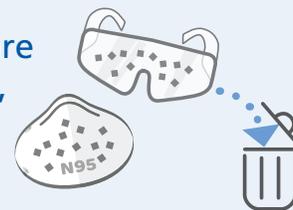

1

Remove gown and gloves inside patient room/area



Perform hand hygiene



Exit patient room/area



Perform hand hygiene, put on new gloves



Place a wipe on table, place eyewear on wipe



2

Remove gloves, perform hand hygiene, put on new gloves



Remove N95 respirator—lower strap first, then upper strap



Store N95 in a paper bag. Mark it with your name, department, and facility.



Remove gloves, perform hand hygiene, put on new gloves



3


Wipe down eyewear and shields
 • front • bands
 • back • ear pieces



Place eyewear and shield upside down on clean table to dry.



Store eyewear and shield in a patient belongings bag.

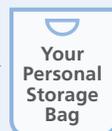

Remove Gloves



Wash hands with soap and water

