



DISINFECTING AND CARING FOR YOUR FACE SHIELD

updated 04/02/20


1 Locate your personal storage bag; place near your designated cleaning area.



Perform hand hygiene; put on new gloves



2 Place a wipe on table; place face shield on wipe, outside down.

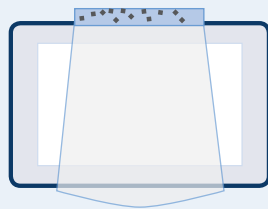

Use a new wipe to disinfect the inside of the face shield.

Replace the foam strip if soiled or loose.

Turn the shield over.



To disinfect, follow wet contact guidelines: purple-top wipes = 2 min., gray-top wipes = 3 min., orange-top wipes = 4 min.



3 Use a new wipe to disinfect the outside of the face shield.



Use a new wipe to disinfect the wipe container and the designated cleaning surface.



4 Remove gloves



Wash hands with soap and water



Place clean shield in your personal storage bag, label it, and store in the designated area.

