



# REPLACING THE FOAM AND ELASTIC ON YOUR FACE SHIELD

updated 04/04/20

If the foam or elastic becomes loose on your face shield, do not throw away the shield. It can be repaired. Please follow the repair process below:

1

Place the clean, disinfected face shield face down on a clean surface.




Perform hand hygiene.



2

Untie the elastic on one side of the shield.

Untie both sides if you are replacing the elastic completely.

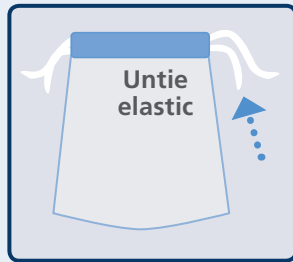

Pull the old foam off of the face shield and discard.



Remove the old 3M tape by rolling it from one side to the other.



3

Place a new 11-inch strip of 3M tape on the shield.

Press firmly to shield, working from one end to the other.



Place the new foam on the 3M tape, pressing it firmly to the shield.

Hold for 30 seconds.



4

Re-tie the elastic on the one side.  
Re-tie on both sides if you are replacing it completely.

Wash hands with soap and water



Place clean shield in your personal storage bag, label it, and store in the designated area.

